

# **RHINO WRIST AND FOREARM SUPPORT**

## Instructions For Use



**MANUFACTURER:**  
MD Orthopaedics, Inc.  
604 North Parkway Street  
Wayland, Iowa 52654 USA  
[www.mdorthopaedicsifu.com](http://www.mdorthopaedicsifu.com)



Instructions For Use  
**RHINO WRIST AND FOREARM SUPPORT**

Published: 2024

### DESCRIPTION

The Wrist & Forearm Support is a medical device intended for use with children and adolescents receiving treatment for medical conditions that impact on the wrist, such as sprains and strains, and as instructed by a healthcare provider. A consistent examination scheduled with the clinical provider is needed to ensure proper fit, use, and instruction for caregivers and/ or users. Compliance with instructions for use and clinical care treatment plan is important to achieve clinical and therapeutic outcomes.

### MATERIALS

The Rhino Wrist and Forearm Support is made from polyester.

### INDICATIONS FOR USE

The Rhino Wrist and Forearm Support is intended for use with children and adolescents receiving treatment for medical conditions that impact on the wrist, such as sprains and strains, and as instructed by a healthcare provider.

### WARNINGS AND PRECAUTIONS

- Follow the qualified healthcare provider and physician instructions or treatment plan.
- Follow instructions to ensure straps are tightened properly, to reduce risk of irritation, blisters, and pressure ulcers.
- Consult your healthcare provider immediately if the child is experiencing pain, irritation, blisters, swelling, pressure ulcers, or if the brace is not fitting properly.
- Skin contact that may cause pain, swelling, allergic or sensory reactions. Do not use Rhino Wrist and Forearm Support over open wounds.
- The Rhino Wrist and Forearm Support should be used without additional components or accessories, unless instructed by the healthcare provider.
- Do not use a Rhino Wrist and Forearm Support that is damaged, broken or not functioning properly.
- Modification is not recommended for the Rhino Wrist and Forearm Support.

### MRI SAFETY INFORMATION

The Rhino Wrist and Forearm Support brace is MR safe.

### CLEANING INSTRUCTIONS

- Wash on delicate cycle or hand wash with mild detergent. Rinse thoroughly.

### STORAGE AND HANDLING

- The Rhino Wrist and Forearm Support requires no special handling during transport or storage.
- The Rhino Wrist and Forearm Support is a nonsterile device.

### APPLYING THE BRACE



1. Open the wrist and forearm support completely. The side that the straps are attached to should be on the thumb side of the arm. Fold the side that does not have straps over the injured arm.



2. Fold the three horizontal velcro straps over the injured arm and attach them to the other side of the splint. Fold the diagonally oriented strap over the hand, between the thumb and index finger, and attach it to the splint.

Instructions For Use  
**RHINO WRIST AND FOREARM SUPPORT**

Published: 2024



**REUSE LIFE**

All parts were designed to be used by a single patient under normal use conditions and as instructed by a healthcare provider. Normal use includes the daily application of the Rhino Wrist and Forearm Support as healthcare provider instructed.

**SAFE DISPOSAL**

The Rhino Wrist and Forearm Support can be disposed of in the regular trash. None of the components of the device are recyclable.

**IMPORTANT STATEMENT**

- MD Orthopaedics, Inc. does not provide medical treatment, advice or recommendations about the risks and benefits of medical treatment, including treatment that involves the use of MD Orthopaedics, Inc. products. This information should be provided solely by the physician or other qualified health care provider treating your child. If you have questions about your child’s treatment, it is important for you to discuss those questions with the appropriate health care provider.
- The manufacturer and distributor are not liable for cases of material damage or personal injury caused by incorrect handling or non-compliance with instructions. Normal use is defined as a single user following intended use.
- Instructions for Use (IFU) are available in English at <https://opsb.com/ifu>. To obtain a copy of the paper IFU, please call Customer Service at MD Orthopaedics, Inc. at (877) 766-7384.
- If a user and/or patient experiences any serious incident that has occurred in relation to the device, it should be reported to MD Orthopaedics, Inc.

**SYMBOL LEGEND**

Symbol	Meaning	Symbol	Meaning
	Manufacturer		Date of manufacture
	Catalog number		Batch code
	Single Patient, Multiple Use		Non-sterile
	Consult Instructions for Use or Consult Electronic Instructions for Use		MR Safe