INSTRUCTION FOR USE
Rhino Cruiser Hip Abduction Brace
Published: 2024

Product Description

The Rhino Cruiser Hip Abduction Brace is a semi-rigid abduction orthosis. The Cruiser holds the hips in the correct abduction position and is flexible enough to allow for crawling or walking. It can be used full-time, or for nighttime wear only (as instructed by a healthcare provider).

The Rhino Cruiser Hip Abduction Brace is frequently used after the Rhino Kicker Pavlik Harness Hip Abduction Brace in pediatric patients who have outgrown the Kicker, are crawling and walking, or who have fixed soft-tissue contractures and fixed hip dislocation. The Rhino Hip Abduction Brace (Cruiser) may also be used when the Kicker has failed to produce a stable reduction.

Materials

The materials of construction are polypropylene, stainless steel, nylon, and foam.

Intended Use

The Rhino Cruiser Hip Abduction Brace is a semi-rigid abduction orthosis, which is intended for treatment of idiopathic acetabular dysplasia and developmental dysplasia of the hip (DDH) and as instructed by a healthcare provider.

Indications for Use

The Rhino Cruiser Hip Abduction Brace is a semi-rigid abduction orthosis, which is intended for treatment of idiopathic acetabular dysplasia and developmental dysplasia of the hip (DDH) in pediatric patients ages three years and younger and as instructed by a healthcare provider.

Clinical Benefits

The Rhino Cruiser Hip Abduction Brace is used to retain the hip in a 90 to 110 degrees of flexion, as instructed by the physician or clinical care provider and should be utilized until the clinical providers instruct that the device is no longer needed.

Evidence demonstrates effectiveness with varied length of daily use and duration based on the patient's clinical care needs. Compliance with instructions for use and clinical care treatment plan is important to achieve clinical and therapeutic outcomes.

Contra-indications

- Do not use a Rhino Cruiser Hip Abduction Brace that is damaged, broken or not functioning properly.
- Modification is not recommended for the Rhino Cruiser Hip Abduction Brace.

Warnings

- Follow the qualified health care provider and physician instructions or treatment plan.
- Follow instructions to ensure straps are tightened properly, to reduce risk of irritation, blisters, and pressure ulcers.
- Consult your physician or clinical team immediately if the child is experiencing pain, irritation, blisters, swelling, pressure ulcers, or if the brace is not fitting properly.
- Protective barriers (i.e. clothing) should be worn when wearing the Rhino Cruiser Hip Abduction Brace.
- Do not apply the brace directly to the skin. Use protective barrier/ clothes between stirrups and skin (e.g., long stretchy pants, lightweight pajamas, etc.).

INSTRUCTION FOR USE
Rhino Cruiser Hip Abduction Brace
Published: 2024

- The brace should be removed to place the child in car seat if the child and brace do not fit in the care seat together.
- The Rhino Cruiser Hip Abduction Brace should be used without additional components or accessories, unless instructed by clinical provider.

MRI Safety Information

The Rhino Cruiser Hip Abduction Brace is MR unsafe and should remain outside the MRI scanner room.

Cleaning Instructions

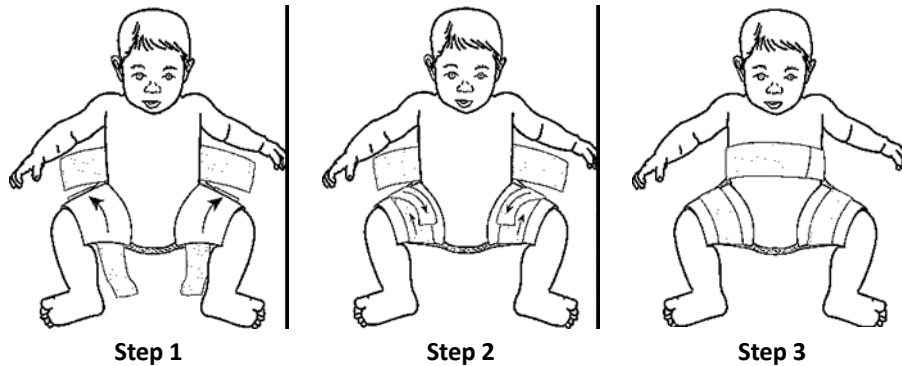
Washing the foam is best accomplished with a washcloth and a body soap that your child is used to. We recommend wiping the brace down 2 to 3 times a week or more frequently as needed.

Do not put the brace in the washing machine or dryer, this will destroy the brace.

Preparation

The Rhino Cruiser Hip Abduction Brace is shipped at room temperature and requires no special handling during transport or storage. The Rhino Cruiser Hip Abduction Brace is a nonsterile device.


Instructions for Applying the Brace

1. Lay the child on their back, lift feet until you are able to slide the brace under the child. Make sure the cross bar is as close to the child as possible. One leg at a time, tug the foam snugly around the thigh, tucking into the back.
2. Next fasten the Velcro straps. There should be no gaps between the thigh and the foam padding.
3. Loosely fasten the waist strap.

Note: Sometimes the foam padding around the legs slides out when the child is walking. Two Self-gripping fasteners can help to keep the foam padding in place.

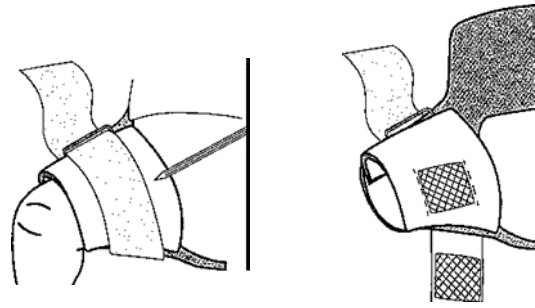INSTRUCTION FOR USE
Rhino Cruiser Hip Abduction Brace
Published: 2024

Step 1

Step 2

1. Slide the strap through the buckle, leaving unbuckled. Mark strap alignment on foam with pencil.
2. Move strap away and apply one side of the fastener to foam within pencil lines. Apply the other side of the fastener to the strap so that both pieces line up when strap is secured. Once applied any change will damage the foam.

Cleaning Instructions

- Washing the foam is best accomplished with a washcloth and a body soap that your child is used to.
- We recommend wiping the brace down 2 to 3 times a week or more frequently as needed.
- Do not put the brace in the washing machine or dryer, this will destroy the brace.

Safe Disposal

The Rhino Cruiser Hip Abduction Brace can be disposed of in the regular trash. None of the components of the device are recyclable.

Storage and Handling

- The Rhino Cruiser Hip Abduction Brace is shipped at room temperature and requires no special handling during transport or storage.
- The Rhino Cruiser Hip Abduction Brace is a nonsterile device.

Reuse Life

All parts were designed to be used by a single patient under normal use conditions and as instructed by a clinical provider. Normal use includes the daily application of the Rhino Cruiser Hip Abduction Brace as clinical provider instructed.

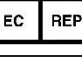
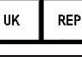
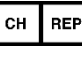
Important Statements

- MD Orthopaedics, Inc. does not provide medical treatment, advice or recommendations about the risks and benefits of medical treatment, including treatment that involves the use of MD Orthopaedics, Inc. products. This information should be provided solely by the physician or other qualified health care provider treating your child. If you have questions about your child's treatment, it is important for you to discuss those questions with the appropriate health care provider.

INSTRUCTION FOR USE
Rhino Cruiser Hip Abduction Brace
Published: 2024

- The manufacturer and distributor are not liable for cases of material damage or personal injury caused by incorrect handling or non-compliance with instructions. Normal use is defined as a single user following intended use.
- Instructions for Use (IFU) manuals are available in English and other languages at <https://opsb.com/> (Regulation (EU) 207/2012). To obtain a copy of the paper IFU, please call Customer Service at MD Orthopaedics, Inc. at (877) 766-7384
- If a user and/or patient experiences any serious incident that has occurred in relation to the device, it should be reported to MD Orthopaedics, Inc. and the competent authority of the EU and EEA Member State in which the user and/or patient is established.

Symbol Legend

Symbol	Meaning	Symbol	Meaning
	Manufacturer		Date of manufacture
	Catalog number		Batch code
	Single Patient, Multiple Use		Non-sterile
	Consult Instructions for Use or Consult Electronic Instructions for Use		Medical Device
	CE Mark Indicates medical device conformity with the provisions of this Directive to enable the device to move freely within the Community and to be put into service in accordance with their intended purpose		Importer
	Authorized Representative in the European Community		UK Representative
	Authorized Representative in Switzerland		MR Unsafe

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