INSTRUCTION FOR USE
Rhino Kicker Pavlik Harness Hip Abduction Brace
Published: 2024

Product Description

The Rhino Kicker Pavlik Harness Hip Abduction Brace consists of a chest strap, two shoulder straps, and two stirrups. Each stirrup has flexion and abduction straps. The Kicker has a biomechanical correct strap position for maximum effectiveness in treating developmental dysplasia of the hip (DDH). Sizes are offered that are suitable for preemie (4-6lbs) through Large (5 months and older).

Materials

The materials of construction are nylon, polyester, and foam.

Intended Use

The Rhino Kicker Pavlik Harness Hip Abduction Brace is a dynamic flexion-abduction orthosis, which is intended for treatment of a developmental dysplasia of the hip (DDH) and as instructed by a healthcare provider.

Indications for Use

The Rhino Kicker Pavlik Harness Hip Abduction Brace is a dynamic flexion-abduction orthosis, which is intended for treatment of a developmental dysplasia of the hip (DDH) in pediatric patients from birth to approximately 6 months of age and as instructed by a healthcare provider.

Clinical Benefits

The Rhino Kicker Pavlik Harness Hip Abduction Brace is used in the treatment to correct DDH deformity by retaining the hip in a 90 to 110 degrees of flexion, as instructed by the physician or clinical care provider and should be utilized until the clinical providers instructs that the device is no longer needed.

Contra-indications

- Do not use a Rhino Kicker Pavlik Harness Hip Abduction Brace that is damaged, broken or not functioning properly.
- Modification is not recommended for the Rhino Kicker Pavlik Harness Hip Abduction Brace.

Warnings


- Follow the qualified health care provider and physician instructions or treatment plan.
- Follow instructions to ensure stirrups/ straps are tightened properly, to reduce risk of irritation, blisters, and swelling.
- Consult your physician or clinical team immediately if the child is experiencing pain, irritation, blisters, swelling, pressure ulcers, or if the brace is not fitting properly.
- Protective barriers (i.e. clothing) should be worn when wearing the Rhino Kicker Hip Abduction Brace.
- Do not apply the brace directly to the skin. Use protective barrier/ clothes between stirrups and skin (e.g., long knee socks).
- The clinical provider will provide instructions for when the harness may be removed for bathing.
- The harness should NOT be removed for diaper changes, unless instructed by clinical provider.
- The Rhino Kicker Pavlik Harness Hip Abduction Brace should be used without additional components or accessories, unless instructed by clinical provider.

MRI Safety Information


The Rhino Kicker Pavlik Harness Hip Abduction Brace is MR safe.

INSTRUCTION FOR USE
Rhino Kicker Pavlik Harness Hip Abduction Brace
Published: 2024

Instructions for Applying the Brace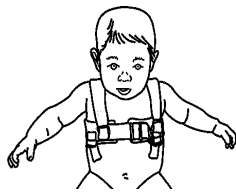

Step 1

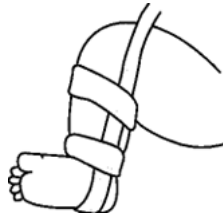Step 2

Step 3

Step 4

1. Position the chest halter at the nipple line and fasten with the Velcro closure.
2. Place leg and foot into the stirrup and fasten the Velcro closure.
3. Attach the front stirrups straps to the outer loops on the front of the halter.
 - a. The straps are adjusted so the hip is flexed as instructed by your physician or care provider.
4. The back stirrup straps are attached to the outer loops on the back of the halter.
 - a. Do not over-tighten these straps.

Notes:

- The Kicker should be worn 23 hours per day until stability is attained.
- A consistent examination scheduled with the clinical provider is needed to ensure proper fit, use, and instruction for caregivers. Compliance with instructions for use and clinical care treatment plan is important to achieve clinical and therapeutic outcomes.
- The hip is placed in 90 to 110 degrees (as instructed by the physician or clinical provider) of flexion and the anterior flexion strap is tightened to maintain position. The knee placement should follow the clinical care plan.

Cleaning Instructions

- Mark strap adjustment points with safety pins.
- Hand or machine wash, delicate cycle; lukewarm water, mild detergent.
- Do not bleach.
- Tumble dry, low heat.

Safe Disposal

The Rhino Kicker Pavlik Harness Hip Abduction Brace can be disposed of in the regular trash. None of the components of the device are recyclable.

Storage and Handling

- The Rhino Kicker Pavlik Harness Hip Abduction Brace is shipped at room temperature and requires no special handling during transport or storage.
- The Rhino Kicker Pavlik Harness Hip Abduction Brace is a nonsterile device.

INSTRUCTION FOR USE
Rhino Kicker Pavlik Harness Hip Abduction Brace
Published: 2024


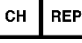
Reuse Life

All parts were designed to be used by a single patient under normal use conditions and as instructed by a clinical provider. Normal use includes the daily application of the Rhino Kicker Pavlik Harness Hip Abduction Brace as clinical provider instructed.

Important Statements

- MD Orthopaedics, Inc. does not provide medical treatment, advice or recommendations about the risks and benefits of medical treatment, including treatment that involves the use of MD Orthopaedics, Inc. products. This information should be provided solely by the physician or other qualified health care provider treating your child. If you have questions about your child’s treatment, it is important for you to discuss those questions with the appropriate health care provider.
- The manufacturer and distributor are not liable for cases of material damage or personal injury caused by incorrect handling or non-compliance with instructions. Normal use is defined as a single user following intended use.
- Instructions for Use (IFU) manuals are available in English and other languages at <https://opsb.com/> (Regulation (EU) 207/2012). To obtain a copy of the paper IFU, please call Customer Service at MD Orthopaedics, Inc. at (877) 766-7384.
- If a user and/or patient experiences any serious incident that has occurred in relation to the device, it should be reported to MD Orthopaedics, Inc. and the competent authority of the EU and EEA Member State in which the user and/or patient is established.

Symbol Legend

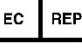Symbol	Meaning	Symbol	Meaning
	Manufacturer		Date of manufacture
	Catalog number		Batch code
	Single Patient, Multiple Use		Non-sterile
	Consult Instructions for Use or Consult Electronic Instructions for Use		Medical Device
	CE Mark Indicates medical device conformity with the provisions of this Directive to enable the device to move freely within the Community and to be put into service in accordance with their intended purpose		Importer
	Authorized Representative in the European Community		UK Representative
	Authorized Representative in Switzerland		MR safe

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