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1. DEVICE DESCRIPTION

The Universal Neonatal Foot Orthotics is a premolded corrective foot sandals for newborns, with stripping providing 1 point fixing foot brace with free ankle joint. It offers an easy to use and time saving treatment.

2. GENERAL INFORMATION

Read all of these instructions carefully because they contain important information about the use, indication, contraindications, application, care and disposal of the UNFO-s. Keep these instructions for further use and review. If you have any question please contact the babies' doctor or our technical service department

3. INDICATIONS FOR USE

The UNFO-s is intended to be worn on an infant foot in order to repair Metatarsus Adductus (MTA).

4. CONTRAINDICATIONS

There are no known medically significant hypersensitivities. With the following medical problems you should use the product after consulting with your doctor:

- Skin problems/injuries to the foot and ankle, especially if there are signs of inflammation. Also, raised scars with swelling, redness and overheating
- Sensation/movement disorders in the leg and feet
- Lymphatic drainage problems, including undiagnosed soft tissue swelling in leg and foot

5. WARNINGS AND PRECAUTIONS

Early treatment in newborn's foot anomalies is essential for better results. Always wear the UNFO-s on sandal or stock footings. Do not put the UNFO-s on too tightly or it could create localized pressure symptoms. In rare cases constriction of blood vessels and nerves is possible. In case of UNFO-s sandal is deform, do not use and please refer to the caregiver.

6. PRINCIPLE OF OPERATION

The UNFO-s should be fitted and approved by the paediatrician or orthopaedic specialist for the first time. The doctor should make a sign over the strap for optimal stretching force. The parents should follow the instructions accordingly. The UNFO-s should be stopped for one hour every day or for half an hour twice a day, permitting to wash the baby and changing the stockings and checking the babies' foot. We advise checking the babies' foot and device every two weeks by the treating doctor.

The UNFO-s should be used for 1 week of daily use despite of fast correction of the foot deformity. Early discontinuation would cause a high rate of recurrence. We advise maintenance usage of UNFO-s for more 1 weeks just for night time. If you notice any unusual behaviour or pressure sores please stop wearing the UNFO-s and inform the doctor. The manufacturer is exempt from product liability if the product is used improperly. Do not make any changes to this product. If this restriction is not observed, the product's performance may be compromised so that product liability is excluded.

7. ADVERSE EVENTS AND THE RISKS RELATED TO THE UNFO-s ARE AS FOLLOWING:

* Targeted pressure that can be caused to the foot. This pressure can be eliminated by releasing the UNFO-s straps.

* The UNFO-s is easy to wear and this might lead to parent's disregard of instruction of use, which may cause an early removal of the UNFO that may lead to MTA recurrence.

* The MTA may not be corrected.

* There may be signs of pressure, blisters and redness - can be eliminated by releasing the UNFO-s straps.

* Allergic reaction - the UNFO-s is worn over stockings, but allergic reactions can always occur.

8. CARE INSTRUCTIONS

Note: Never expose the UNFO-s to direct heat. This could damage the materials. Don't wash the device in washing machine avoiding damage to the UNFO-s device. Clean the device with wet tissue and mild laundry detergent.

9. INSTRUCTIONS FOR USE

The UNFO-s is composed of the following: A. Supporting shell B. Cushioned strap 9.1 Putting the sandals on

There are two different devices for feet, right and left. Be sure that you use the appropriate device for the deformed foot.

* Wear on a soft cotton rich stocking before putting on the UNFO-s device. The stocking is supposed to avoid skin maceration and laceration and acts as a sweat absorbing pad.

* Be sure that the cushioned strap is open and loose and attaches to the inside of the supporting shell.

* Put on the UNFO-s with the cut out of the sole is fronted to the outside of the foot, the heel inside the heel cage and the big toe covered beneath the big toe support.

* The cushioned strap is wrapped around the foot toward the outside, passes through the cut out and under the sole of the UNFO-s making a loop over the strap underneath. Fasten the strap, wrap it back and attach on itself by Velcro on the back of the foot.


10. INFORMATION ABOUT RE-USE This product is single patient use. The UNFO-s is intended for your use only, so do not give the UNFO-s to anyone else.

11. WARRANTY

This product is guaranteed in accordance with statutory regulations and UNFO-s AG's general terms and conditions (GTC) if all of the aforementioned points are adhered to.

12. DISPOSAL

You can dispose of this product with your household garbage.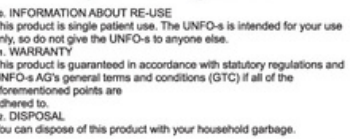

REF	LOT	YYYY-MM-DD	Single Use Only
Model Number	Lot Number		
Refer to instructions	Do not use if package is damaged	Keep away from sunlight	Keep dry

MANUFACTURED BY: UNFO Med Ltd. 52 Wakefield Street, Hoin, 58326 Israel. Tel: +972-3-5010383

Obelis s.a. Bd. Général Waha 53 B-1030 Brussels, Belgium Phone: 32-2-732-59-54 Fax: 32-2-732-60-03

ECNIPROBIOLOGICO IN UNO