

Instructions For Use
RHINO ARM SLING

Published: 2024



RHINO ARM SLING

Instructions For Use



MANUFACTURER:

MD Orthopaedics, Inc.
604 North Parkway Street
Wayland, Iowa 52654 USA
www.mdorthopaedicsifu.com



Instructions For Use RHINO ARM SLING

Published: 2024



DESCRIPTION

The Rhino Arm Sling is an attractive blue envelope sling with an adjustable strap. This sling has self-gripping fasteners that make it very easy to apply. A consistent examination scheduled with the clinical provider is needed to ensure proper fit, use, and instruction for caregivers and/or users. Compliance with instructions for use and clinical care treatment plan is important to achieve clinical and therapeutic outcomes.

MATERIALS

The Rhino Arm Sling is made from cotton, polyester, d-ring arm sling, nickel plated slide, nylon, metal rectangular welded loop, bias white, twill fabric, and polyester low denier filament fiber.

INDICATIONS FOR USE

The Rhino Arm Sling is intended for use with pediatric patients to immobilize the arm by means of a fabric band suspended from around the neck and as instructed by a healthcare provider.

WARNINGS AND PRECAUTIONS

- Follow the qualified healthcare provider and physician instructions or treatment plan.
- Follow instructions to ensure straps are tightened properly, to reduce risk of irritation, blisters, and pressure ulcers.
- Consult your healthcare provider immediately if the child is experiencing pain, irritation, blisters, swelling, pressure ulcers, or if the brace is not fitting properly.
- Skin contact that may cause pain, swelling, allergic or sensory reactions. Do not use Rhino Arm Sling over open wounds.
- The Rhino Arm Sling should be used without additional components or accessories, unless instructed by the healthcare provider.
- Do not use a Rhino Arm Sling that is damaged, broken or not functioning properly.
- Modification is not recommended for the Rhino Arm Sling.

MRI SAFETY INFORMATION

The Rhino Arm Sling brace is MR unsafe and should remain outside the MRI scanner room.


CLEANING INSTRUCTIONS

- Wash on delicate cycle or hand wash with mild detergent. Rinse thoroughly.


STORAGE AND HANDLING

- The Rhino Arm Sling requires no special handling during transport or storage.
- The Rhino Arm Sling is a nonsterile device.

APPLYING THE RHINO ARM SLING

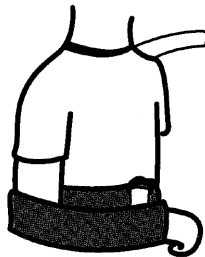

1. Carefully seat the elbow of the injured arm in the enclosed corner of the sling. Send the strap over the shoulder opposite the injured arm.

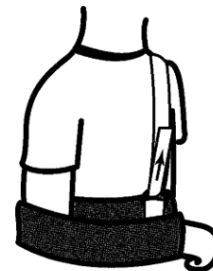

2. Thread the strap through the metal loop near the hand and Velcro the strap onto itself.

Instructions For Use RHINO ARM SLING

Published: 2024



REUSE LIFE

All parts were designed to be used by a single patient under normal use conditions and as instructed by a healthcare provider. Normal use includes the daily application of the Rhino Arm Sling as healthcare provider instructed.

SAFE DISPOSAL

The Rhino Arm Sling can be disposed of in the regular trash. None of the components of the device are recyclable.

IMPORTANT STATEMENT

- MD Orthopaedics, Inc. does not provide medical treatment, advice or recommendations about the risks and benefits of medical treatment, including treatment that involves the use of MD Orthopaedics, Inc. products. This information should be provided solely by the physician or other qualified health care provider treating your child. If you have questions about your child's treatment, it is important for you to discuss those questions with the appropriate health care provider.
- The manufacturer and distributor are not liable for cases of material damage or personal injury caused by incorrect handling or non-compliance with instructions. Normal use is defined as a single user following intended use.
- Instructions for Use (IFU) are available in English at <https://opsb.com/ifu>. To obtain a copy of the paper IFU, please call Customer Service at MD Orthopaedics, Inc. at (877) 766-7384.
- If a user and/or patient experiences any serious incident that has occurred in relation to the device, it should be reported to MD Orthopaedics, Inc.

SYMBOL LEGEND

Symbol	Meaning	Symbol	Meaning
	Manufacturer		Date of manufacture
	Catalog number		Batch code
	Single Patient, Multiple Use		Non-sterile
	Consult Instructions for Use or Consult Electronic Instructions for Use		MR Unsafe